

# HemaPRO slide Stainer

Automated Blood Smear Staining



 Online Ordering Available

**QUICKSLIDE™**  
A DIVISION OF HARDY DIAGNOSTICS

# INTRODUCING

## QUICKSLIDE™ HEMAPRO

Easy-to-use LCD display

Advanced microprocessor increases speed and ease of operator use


Soft touch color-coded keypad

Computer monitors reagent and prompts when to reorder more stain

Adjustable stain and buffer timing – important for consistent lab results with multiple users

Color-coded reagent tubing for easy of set-up

Insert slide here

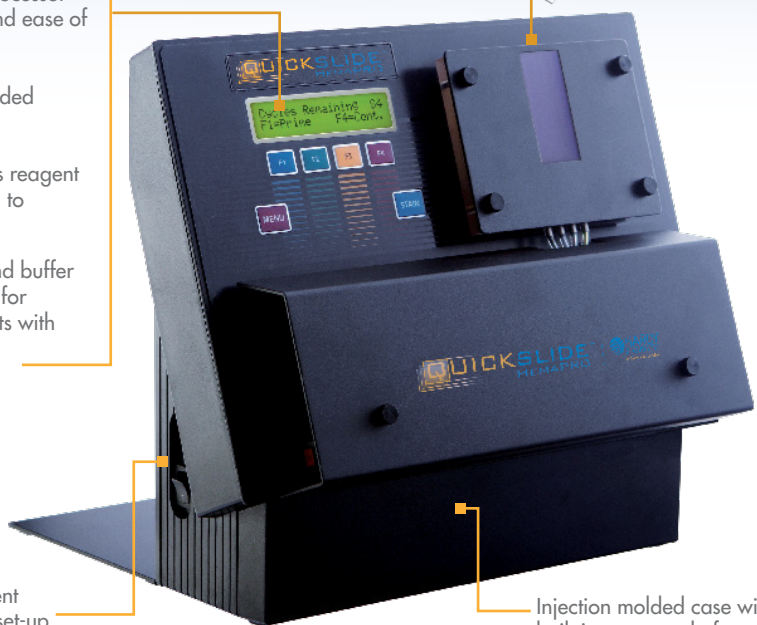

Injection molded case with built-in reagent platform and small footprint

The HemaPRO™ Slide Stainer represents a significant advancement in automated blood smear staining. It is a valuable addition to any laboratory, whether it is the primary stainer used by small hospitals, physicians labs, stat labs, or used as a statim or backup instrument in larger labs. The HemaPRO™ can also be utilized in veterinary labs for a variety of biological specimens.

# BIG FEATURES. SMALL SIZE.

### No hassle.

The unit needs no daily maintenance and only requires a tubing change once or twice annually.

### Consistent.

The HemaPRO™ yields consistent results no matter who stains the slide. Very little training is needed.

### One size fits all.

The HemaPRO™ is one of the smallest automated slide stainers available, requiring only one square foot of counter space.

### Fast staining.


The HemaPRO™ can stain a blood smear in as little as 60 seconds.

### Versatile.

In addition to staining peripheral blood smears, the HemaPRO™ has a special program that allows it to stain bone marrow samples. The HemaPRO™ also allows the operator to adjust both stain and buffer times.

Catalog no. QS-001

## Replacement Tubing

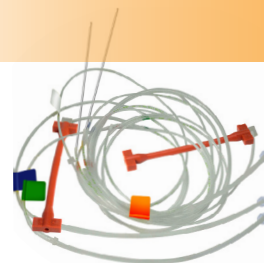

### HemaPRO™ Tube Kit

Kit consists of labeled replacement tubing. Replacement should be performed every 6 to 12 months depending on use. Tubing can be installed in just 10 minutes.

Each..... QS-002

## Reagents & Cleaners



### HemaPRO™ Reagent Kit

Kit contains Wrights-Giemsa stain, Buffer, and Rinse. Produces 125 tests.

3/pk ..... QS-29000



1430 West McCoy Lane  
Santa Maria, CA 93455

[www.HardyDiagnostics.com](http://www.HardyDiagnostics.com) | [Sales@HardyDiagnostics.com](mailto:Sales@HardyDiagnostics.com)  
Manufactured in Wichita Falls, Texas