Pinworm Paddle V302-PIN

Patient Instructions:

1. Collect sample first thing in the morning before washing or using toilet. Or one to three hours after retiring for the evening.
2. Holding the cap, remove the lid and paddle assembly from the vial provided. Do not disconnect the paddle from the lid. **Do Not Touch the Paddle.**
3. Separate the buttocks and press the sticky surface of the paddle against the anal area.
4. Return the lid and paddle assembly to the container and tightly secure the lid.
5. Wash your hands thoroughly with soap after you have finished.
6. Complete the patient information on the container label (patient's name, date of birth, date and time of collection).
7. Keep the specimen at room temperature and deliver to the laboratory as soon as the collection is completed.

Technologists Directions:

1. Remove the cap with the attached paddle.
2. Remove the paddle from the cap, gripping only the stem of the paddle.
3. Place the paddle STICKY SIDE UP on the microscope stage and examine.
4. When examination is completed, return paddle to container, replace cap and dispose of as with any microbiologic specimen.

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