

# We're always looking for ways to improve your customer experience with QuickSlide™!

To reduce the possibility of a slide breaking in the cuvette, here are some helpful tips:

1


Be careful to align properly!



Hold the slide perfectly straight (vertically) when inserting into the cuvette.

Go slow!


2

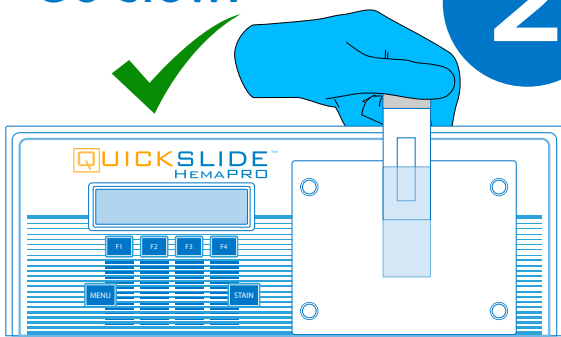

Slowly lower the slide into the cuvette while holding it straight. Once the slide has made contact with the stopper, you may release the slide.

When staining is complete, slowly take the slide out the same way you put it in.

3

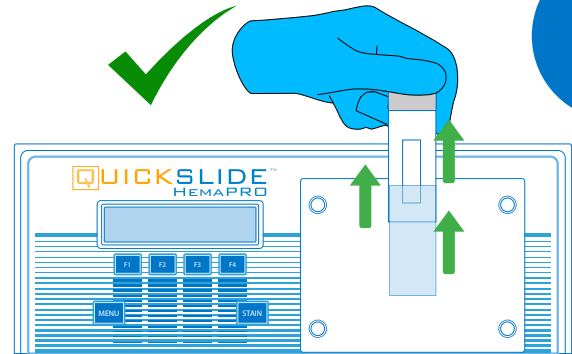

**DO NOT** move or rock the slide back and forth; otherwise the slide may break or chip inside the cuvette.

**Need help? We're standing by!**

Call our Technical Support team for assistance with all of your Hardy products!

**800.266.2222, opt. 2**

 **HARDY**  
DIAGNOSTICS  
A Culture of Service™